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	Patient information and care instructions for implants and abutments
	MDR Article 18

Applicable to the companies	Yes	No
bredent GmbH & Co. KG	☐	☒
bredent medical GmbH & Co. KG	☒	☐

Dear patient,

You have made an excellent choice on deciding to choose the SKY implant system from bredent medical (single registration number (SRN) from the manufacturer DE-MF-000008125). The SKY implant system consists of the implant itself which heals into the jaw bone. In addition, a body that is screwed into the implant and onto which the prosthetic restoration is attached. bredent medical offers various implant systems for different clinical requirements which are specifically described in the implant identification card. The proven products of bredent medical in implantology and prosthetics are the result of their own production in Germany. Decades of research, the knowledge of more than 50 years, and constant dialogue with dentists and dental technicians were incorporated into the development. Clinical and scientific tests of bredent medical products are performed in detail by universities. This is why the SKY implant system is distinguished by its high quality and precision “Made in Germany” and is subject to a quality management system certified in accordance with DIN EN ISO 13485. The dental implants are class IIb medical devices. These products restore the chewing function, social function and phonetic function of the teeth. Clinical success and the associated safety have been proven with well over one million implants which have been successfully inserted and positively evaluated in scientific documentation since 2002. bredent medical offers a lifetime guarantee on SKY implants with original SKY prosthetic abutments. In addition, bredent medical ensures that the necessary parts for the preservation and follow-up care of a SKY implant remain available for many years. We hope you enjoy your implantology treatment and the quality of life you have regained.

You will find important information on the safe handling and care of implants and abutments below. Please read them carefully.

Implant card and patient passport

The identification details of the product including the product name, serial number or lot number, unique device identifier (UDI), reference number, manufacturer’s address and website may be found on the implant card.

This implant card (small sticker) is supplied with every implant and may be stuck onto the patient card (International Implant Card) provided for this purpose. This gives your attending doctor and dental technician a good overview of which products have already been used. This is especially important in case of changing dentists.

Revision	Page number	File name
1	12/2021	1 of 4 DO Patient Information and care instructions for implants and abutments

	Patient information and care instructions for implants and abutments
	MDR Article 18

Explanation of the symbols on the implant or patient ID card.

	Patient Name
	Healthcare facility (treating physician/practice)
	Implantation Date
	Website with information for patients
	Medical Device- Medical product
	Serial number:
	Batch
	Unique Device Identifier – Unique Product Identifier
	Manufacturer

Please feel free to ask your dentist for the patient passport which should be delivered to you.

Notes on healing:

Please note the following instructions to promote healing after the operation, unless you have received a specific recommendation by your dentist.

1. Do not eat or drink until the local anaesthesia subsides. Otherwise you may unnoticedly bite your cheeks.
2. As your reaction ability is impaired after the procedure, you should not drive a vehicle.
3. Cool the wound area in the first hours after the surgery. Do not leave the cooler bag or cold washcloth in one place for more than half a minute to prevent hypothermia.
4. After surgery, avoid excessive consumption of alcohol and caffeinated drinks such as coffee, black tea, green tea and nicotine.
5. Do not expose yourself to any physical exertion in the first few days, especially sports activities, as well as heat, sauna and sunbathing.
6. Avoid smoking until the day of suture removal. Smoking puts the blood supply at risk, this may lead to significant wound healing disorders and spontaneous implant loss.

Revision	Page number	File name
1	12/2021	2 of 4
		DO Patient Information and care instructions for implants and abutments

7. During the hours following the operation, a blood plug forms, which contributes to wound healing. The grey coating is also part of the body's own healing process. Do not remove!
 8. A small amount of bleeding from the wound is normal. You can stop minor secondary bleeding yourself by gently biting a gauze or folded fabric handkerchief in order to apply pressure onto the wound. Major bleeding is rare. If this is the case, please contact the office as soon as possible or contact the dental emergency service.
 9. Avoid excessive stress on the wound area. You can brush your teeth as usual, but protect the wound and gums in the affected area. Do not place fingers on the wound. After eating, rinse mouth with a mouthwash solution or sage tea.
 10. If possible, do not eat fresh dairy products with bacterial cultures (e.g. yogurt) until suture removal. The lactic acid bacteria contained therein may cause infections and wound healing disorders.
 11. It is recommended to eat only predominantly soft food (pasta, rice, vegetables, fish, tender meat, etc.) for 6 weeks. Foods such as nuts, hard bread, tough meat and similar foods which entail a high chewing load must be avoided.
 12. In very rare cases, screw loosening may occur during your implantology treatment or in absolutely exceptional cases, your temporary bridge may be broken. In this case, please contact the office immediately so that the bridge can be retightened or repaired. If this is not done, it is possible that major damage up to loss of implants occurs.
 13. Please take the prescribed medication with specified dosing. If you experience pain, you can also take pain killers at any time. However, please refer to the package insert due to use and compatibility.
- If you experience persistent pain or major secondary bleeding, please go to the dentist office immediately.

Dental care advice:

Implant-supported teeth may be easily cleaned with an (electric) toothbrush and toothpaste, just like your own teeth. In the case of implants, the interproximal spaces and transitions to the gums are particularly important. Bacteria colonise here quickly and cause some complaints such as bleeding gums, decline in gums, bad breath, pain and tooth loosening. In the worst case scenario, this can lead to tooth and implant loss. However, this can be effectively prevented with dental floss or small interdental brushes and regular professional tooth cleaning as well as HELBO®therapy. This is another reason why regular monitoring and cleaning appointments should be held.

Note on complications

Our products are produced and controlled to the highest quality standards. However, complications such as screw loosening, screw breakage may occur. In this case, we recommend that you consult your dentist immediately in order to eliminate this complication and avoid major damage.

Revision	Page number	File name
1	12/2021	3 of 4
DO Patient Information and care instructions for implants and abutments		

In very rare cases, inflammation may occur around the implant, which may be caused by allergic reactions. In such cases, we also recommend that you consult your dentist immediately.

Biological conversion processes are constantly taking place in the mouth, so that the load introduction of the chewing forces can change over time. It is therefore necessary for you to regularly have control appointments; we recommend at least twice a year. Professional cleaning of the product should also be performed during these visits.

Revision	Page number	File name
1	4 of 4	DO Patient Information and care instructions for implants and abutments

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© bredent