


SKY pro guide – overview of drill application

Implant	Pilot drill	Twist drill	Final drill		Crestal drill	Implant mounter	
			Hard bone D1	Medium hard / soft bone D2, D3, D4		SKY	copaSKY
5.2 mm	4.0	PCDXLXD450	TDXL52D225	1. FD1L52D350 2. FD1L52D400	CRDXLXD400	n/a	COPMLXD475
	4.5		TDXL52D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450	CRDXLXD450	n/a	COPMLXD475
8 mm	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL08D225	1. FD1L52D350 2. FD1L08D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	4.0		1. TDXL52D225 2. TDXL08D225	1. FD1L52D350 2. FD1L52D400 3. FD1L08D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5		1. TDXL52D225 2. TDXL08D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L08D450	CRDXLXD450	SKYMLXD475	COPMLXD475
10 mm	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L10D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	3.5		1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L10D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L52D400 3. FD1L10D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5		1. TDXL52D225 2. TDXL10D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L10D450	CRDXLXD450	SKYMLXD475	COPMLXD475

Case study: * bSKY4514

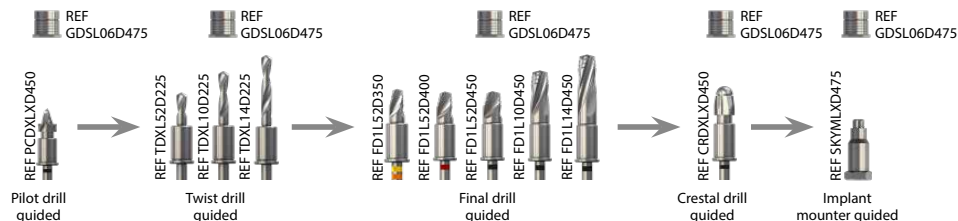

Surgical protocol guided implantology

SKY pro guide – overview of drill application

Implant	Pilot drill	Twist drill	Final drill		Crestal drill	Implant mounter	
			Hard bone D1	Medium hard / soft bone D2, D3, D4		SKY	copaSKY
12 mm	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L12D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	3.5		1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L12D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L52D400 3. FD1L12D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5		1. TDXL52D225 2. TDXL12D225	1. FD1L52D350 2. FD1L52D400 3. FD1L12D450 4. FD2L12D450	CRDXLXD450	SKYMLXD475	COPMLXD475
14 mm	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL10D225 5. TDXL14D225	1. FD1L52D350 2. FD1L10D350 3. FD1L14D350	CRDXLXD35n	SKYMLXD475	COPMLXD475
	3.5		1. TDXL52D225 2. TDXL10D225 5. TDXL14D225	1. FD1L52D350 2. FD1L10D350 3. FD1L14D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL10D225 3. TDXL14D225	1. FD1L52D350 2. FD1L52D400 3. FD1L10D400 4. FD1L14D400	CRDXLXD400	SKYMLXD475	COPMLXD475
	4.5 *		1. TDXL52D225 2. TDXL10D225 3. TDXL14D225	1. FD1L52D350 2. FD1L52D400 3. FD1L52D450 4. FD1L10D450 5. FD1L14D450	CRDXLXD450	SKYMLXD475	COPMLXD475
16 mm	3.5 N	PCDXLXD450	1. TDXL52D225 2. TDXL10D225 3. TDXL16D225	1. FD1L52D350 2. FD1L10D350 3. FD1L16D350	CRDXLXD35n	SKYMLXD475	n/a
	3.5		1. TDXL52D225 2. TDXL10D225 3. TDXL16D225	1. FD1L52D350 2. FD1L10D350 3. FD1L16D350	CRDXLXD350	SKYMLXD475	n/a
	4.0		1. TDXL52D225 2. TDXL10D225 3. TDXL16D225	1. FD1L52D350 2. FD1L52D400 3. FD1L10D400 4. FD1L16D400	CRDXLXD400	SKYMLXD475	n/a

50 years of bredent

bredent
medical

bredent
group

50 years of bredent

bredent
medical


bredent
group



SKY
PRO GUIDE




Case study: copa4014

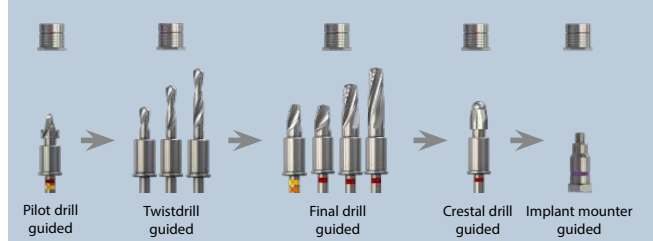


3.5N
3.5
4.0
4.5

Pilot drill guided

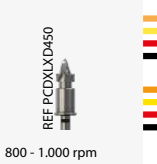

800 - 1.000 rpm

Twist drill guided

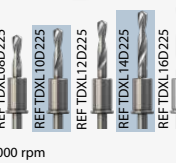

800 - 1.000 rpm

Implant length

8, 10, 12, 14, 16 mm
3.5N 3.5

5, 2, 8, 10, 12, 14, 16 mm
4.0

5, 2, 8, 10, 12, 14, mm
4.5

Implant mounter guided



25 rpm
max. 45 Ncm

Crestal drill guided



300 rpm

Hard bone D1

Final drill guided




300 rpm 3.5N 3.5

Final drill guided




300 rpm 4.0

Final drill guided

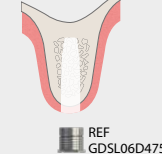


300 rpm 4.5

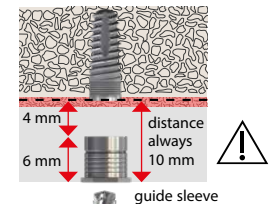

REF GDSL06D475 REF GDSL04D475


Medium hard / soft bone D2, D3, D4

Final drill guided

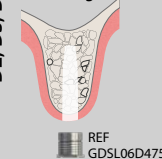


300 rpm 3.5N 3.5

Final drill guided

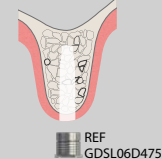

300 rpm 4.0

Final drill guided

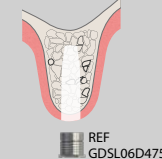

300 rpm 4.5

50 years of brendent

brendent
medical

brendent
group